



# RADTAG®

## Blood Irradiation Indicators

The Leader in Safety  
Verification...Pure and Simple

THE MOST PRECISE BLOOD IRRADIATION INDICATOR

NOW AVAILABLE IN TWO STYLES!

### MINIMUM DOSE VERIFICATION

**RADTAG®**

**IRRADIATION INDICATOR**  
If central dot white, not irradiated ✗  
If central dot blue, irradiated ✓

Date: \_\_\_\_/\_\_\_\_/\_\_\_\_ Lot No.: 9999  
ID: \_\_\_\_ Min 25Gy Exp: Jan 0000


RTX25-S (X-RAY)

**RADTAG®**

**IRRADIATION INDICATOR**  
If central dot white, not irradiated ✗  
If central dot blue, irradiated ✓

Date: \_\_\_\_/\_\_\_\_/\_\_\_\_ Lot No.: 9999  
ID: \_\_\_\_ Min 25Gy Exp: Jan 0000

RTG25-S (GAMMA)

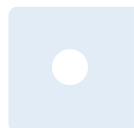

Not Irradiated



Irradiated

### MINIMUM/MAXIMUM DOSE VERIFICATION

**RADTAG®**

**CENTRAL BLUE DOT INDICATES X-RAY IRRADIATION**

MIN 25Gy MAX 50Gy

Date: \_\_\_\_/\_\_\_\_/\_\_\_\_ Lot No.: 9999  
Time: \_\_\_\_ ID: \_\_\_\_ Exp: Jan 0000  
Target Dose ☐ 35 Gy Other: \_\_\_\_ Gy RADTAG®

RTX25

**RADTAG®**

**CENTRAL BLUE DOT INDICATES GAMMA IRRADIATION**

MIN 25Gy MAX 50Gy

Date: \_\_\_\_/\_\_\_\_/\_\_\_\_ Lot No.: 9999  
Time: \_\_\_\_ ID: \_\_\_\_ Exp: Jan 0000  
Target Dose ☐ 35 Gy Other: \_\_\_\_ Gy RADTAG®

RTG25



**Under-Irradiated**

Central Dot lighter than minimum reference color



**Within Recommended Dose Range**

Central Dot between minimum and maximum reference color



**Over-Irradiated**

Central Dot darker than maximum reference color

RadTag® indicators are specially calibrated to verify that the dose of irradiation delivered falls within recommended ranges and that irradiated blood is safe to transfuse.

Many factors can lead to under-irradiation or over-irradiation of blood products. Yes/No indicators cannot reliably tell you if your blood products received the correct dose of irradiation. RadTag® indicators offer the best final validation that irradiated blood is safe to transfuse. It's for this reason leading irradiator manufacturers recommend RadTag®.

Manufactured in compliance with ISO 9001:2015 guidelines



**RADTAG**  
TECHNOLOGIES

2403 - 96 Street NW, Edmonton, Alberta CANADA T6N 0A7  
Web: [www.radtagtech.com](http://www.radtagtech.com) E-mail: [info@radtagtech.com](mailto:info@radtagtech.com) Tel: +1.780.433.4331